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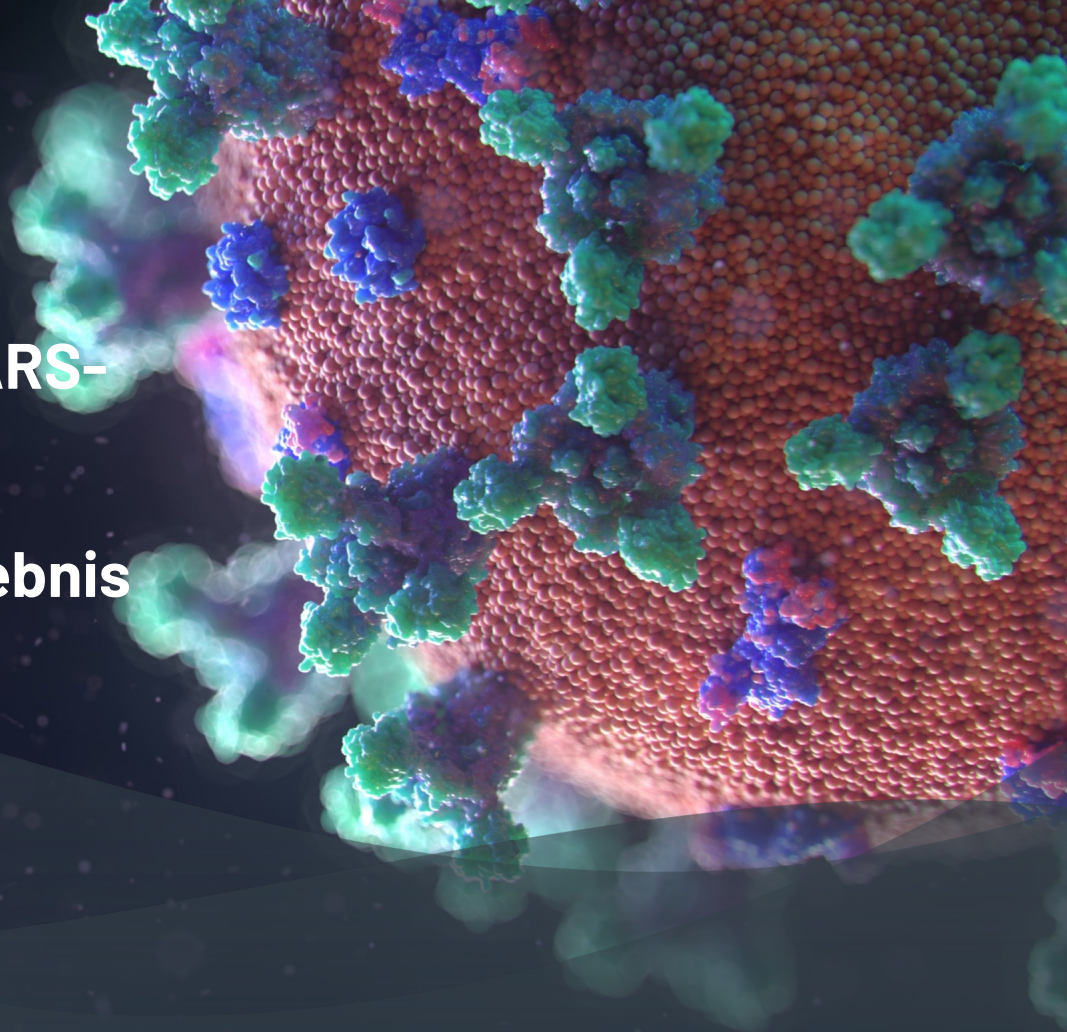
midge medical is the future of diagnostics.


Innovative digital SARS-CoV-2 analytics: 15 minutes to molecular test result in the nephrology center

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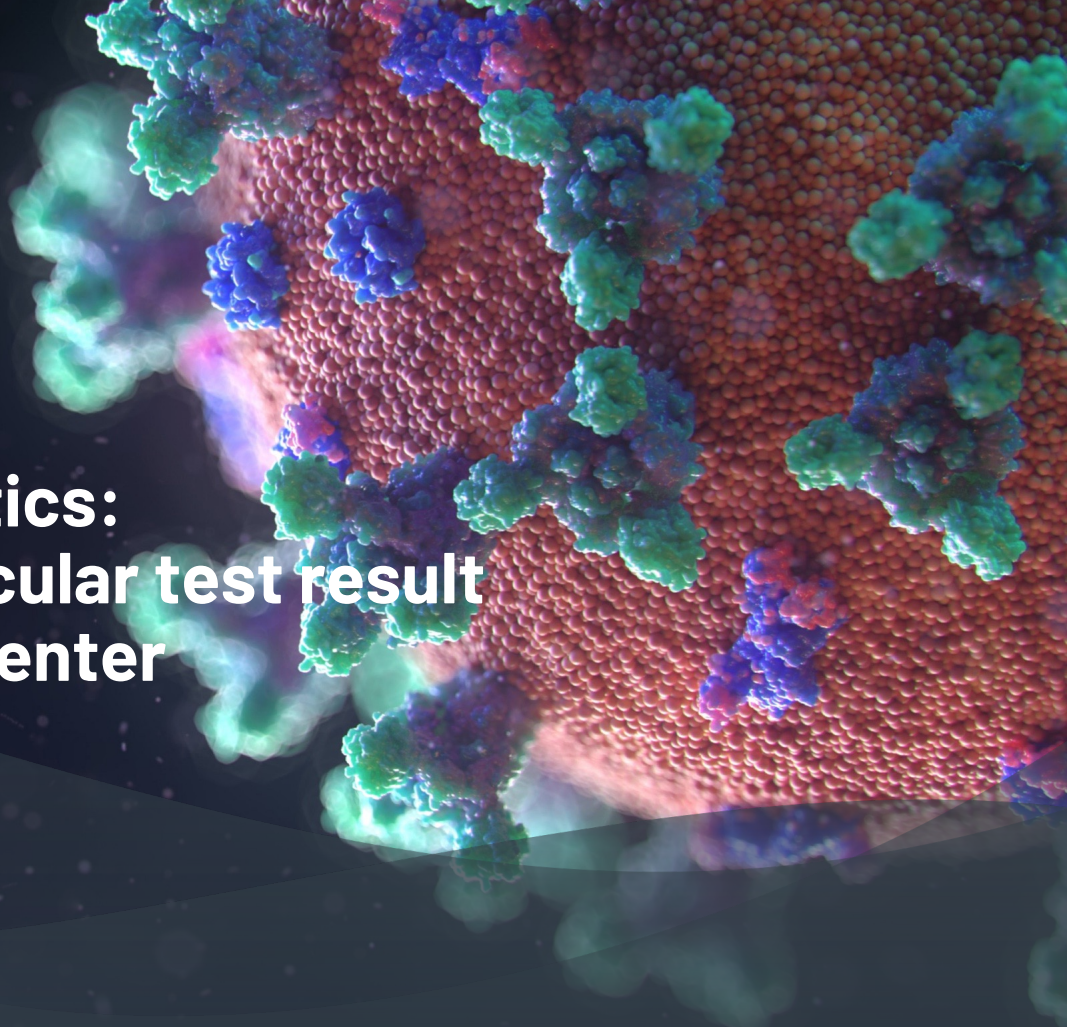midge medical GmbH

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Disclaimer

The following content reflects management's assessment at the time of preparation. There is therefore a certain risk that subsequent findings may invalidate the conclusions drawn. In addition, management's conclusion and reasoning may be inaccurate.

It is therefore the user's responsibility to draw his or her own conclusions and form his or her own opinion about the facts and, in particular, the assessments made. This applies in particular if the user makes his or her own business decisions based on the facts and conclusions presented.

For this purpose, we are happy to provide primary information.

All information in this presentation is provided without any warranty as to its timeliness, accuracy or completeness. midge medical excludes liability for damages that may result directly or indirectly from the use of the presentation and the information contained therein. The limitations of liability shall not apply insofar as the liability is based on intent or gross negligence, as well as for any damages resulting from injury to life, body, health or due to other mandatory legal provisions.

It is pointed out that the SARS-CoV-2 test developed by midge medical GmbH has not yet been approved and is not offered for sale. midge medical GmbH plans to obtain approval of the test as an IVD according to the Medical Devices Act.

The presentation is for the recipient's internal use only, as the information contained therein is confidential. It is only intended for the respective addressee and may only be passed on to third parties in its entirety with the prior written consent of midge medical GmbH.

Questions may be directed to Michael Diebold (michael.diebold@midge-medical.com) or Dr. Markus Riester (markus.riester@midge-medical.com).

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midge medical – handheld diagnostic device with molecular precision Made in Berlin.

Startup founded in Berlin

Goal Develop analytical procedures for bringing fully digital diagnostics to the health care professional

Step 1 Simplified, fast SARS-CoV-2 diagnostics for **nephrologists starting in 20/22**, for high-risk HD patients and ambulatory care

Step 2 2023: Bring diagnostics home for high risk and PD patients, with supervision of results by physicians

Founders

Michael Diebold

Dr. Markus Riester

CKD patients are at high risk in the SARS-CoV-2 pandemic

CHRONIC KIDNEY DISEASE

CKD is a key risk factor for COVID-19 mortality


[Ron T. Gansevoort](#) ✉ & [Luuk B. Hilbrands](#) ✉

[Nature Reviews Nephrology](#) **16**, 705–706 (2020) | [Cite this article](#)

19k Accesses | **39** Citations | **49** Altmetric | [Metrics](#)

A new study uses the OpenSAFELY health analytics platform to identify risk factors for COVID-19 mortality. This analysis, which includes data for more than 17 million people in the UK, suggests that patients with chronic kidney disease are at higher risk than those with other known risk factors, including chronic heart and lung disease.

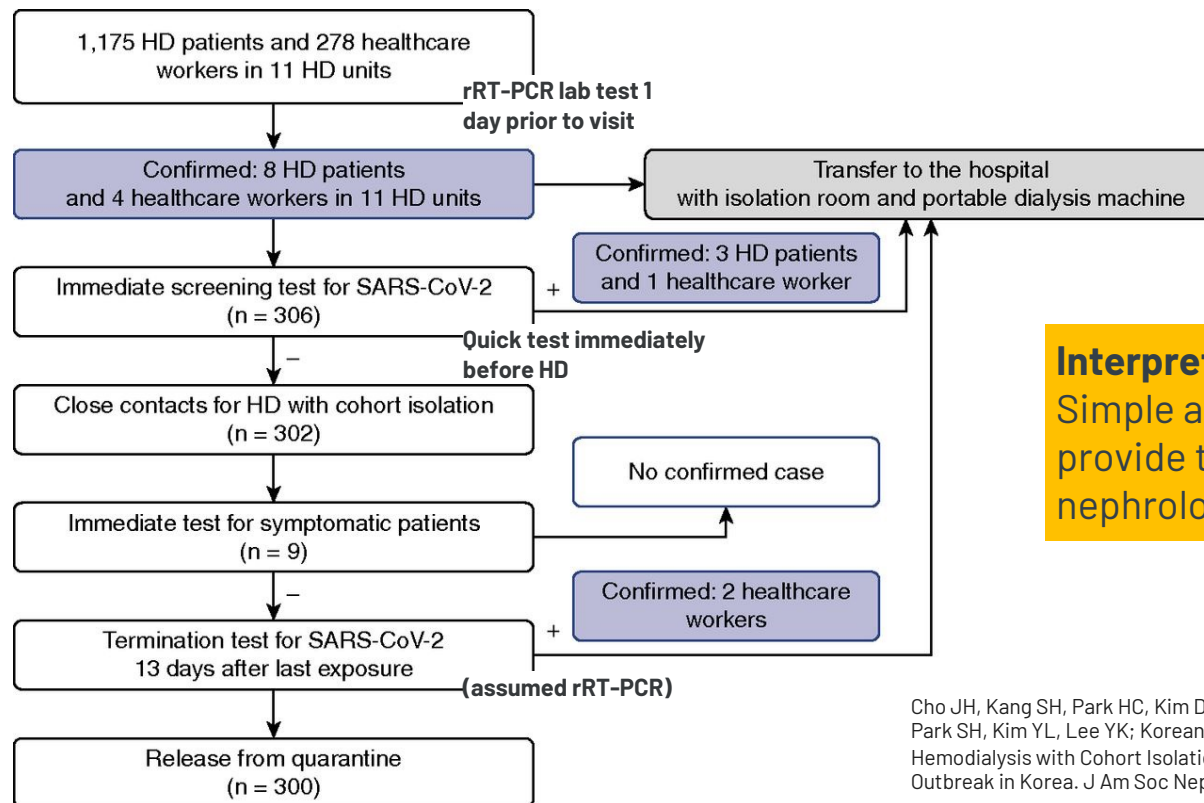Multi-center study determined factors to limit exposure of HD clinics to SARS-CoV-2

Interpretation

Simple and fast molecular test can provide the required safety for the nephrological clinic

Cho JH, Kang SH, Park HC, Kim DK, Lee SH, Do JY, Park JW, Kim SN, Kim MS, Jin K, Kang GW, Park SH, Kim YL, Lee YK; Korean Society of Nephrology COVID-19 Task Force Team. Hemodialysis with Cohort Isolation to Prevent Secondary Transmission during a COVID-19 Outbreak in Korea. J Am Soc Nephrol. 2020 Jul;31(7):1398-1408. doi: 10.1681/ASN.2020040461.


Conclusion: early detection is critical to limit exposure of HD clinics to SARS-CoV-2 infection

Conclusions from study	Short term contribution of midge test	Propects for the clinic
pre-existing deployment plan before the epidemic		PoC molecular diagnostics included in planning
early detection with rapid rRT-PCR testing	Fast molecular test for testing patients, staff, drivers, doctors	Validation test can be done by minoo test in the nephrological clinic
collaboration between institutions	Enabled by digital connectivity, supported by midge test device	Nephrologist controls the diagnostic process to create a safe workplace and improve patient outcomes
continuous monitoring of infection	Enabled by rapid PoC minoo test done at the clinic	Control infections exposure by rapid PoC minoo test done at the clinic

Current patient journey

Today

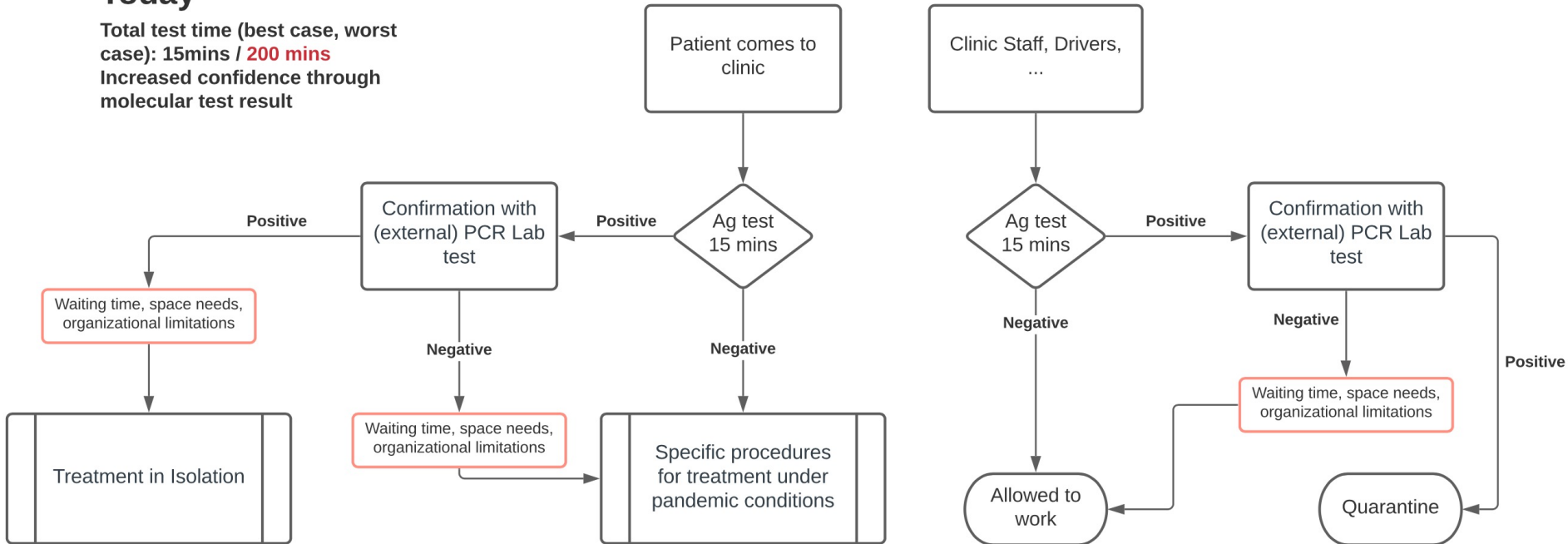
Total test time (best case, worst case): 15mins / **200 mins**
Increased confidence through molecular test result

Future patient journey

minoo molecular PoC test

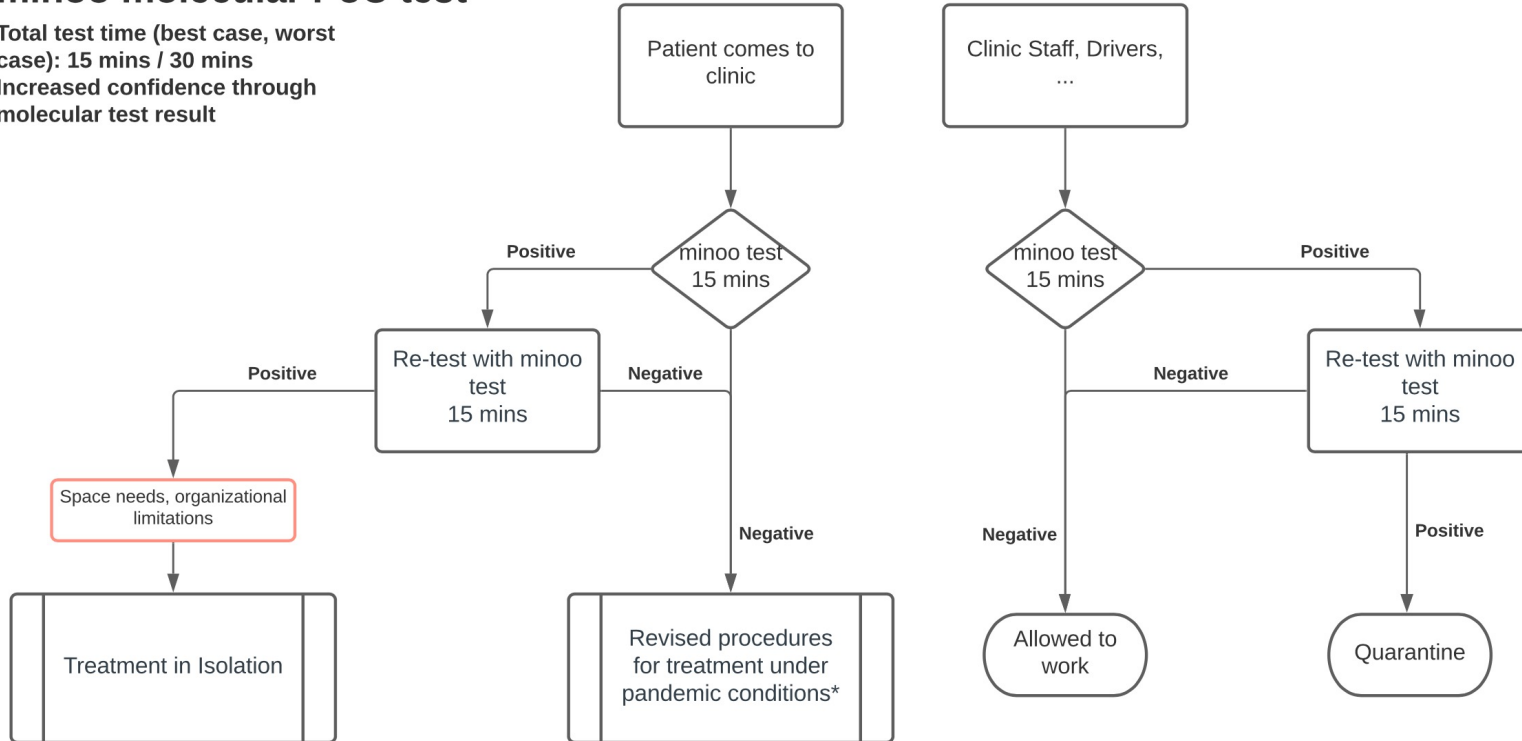Total test time (best case, worst case): 15 mins / 30 mins
Increased confidence through molecular test result

Test capacities again reaches the limits ... time-to-result on the rise

- Same situation as 2020
- In Germany about 2 million tests available per week!
- At least 5-fold amount of tests needed for combating the pandemic ... more is better

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Von Stephan Lenhardt, SWR Mainz

Saving time and reducing costs by speeding up the testing process before patients start their dialysis

Reported mortality rate of 15% due to SARS-COV-2

As a high-risk patient group, they need to get PCR tested every time before starting their treatment

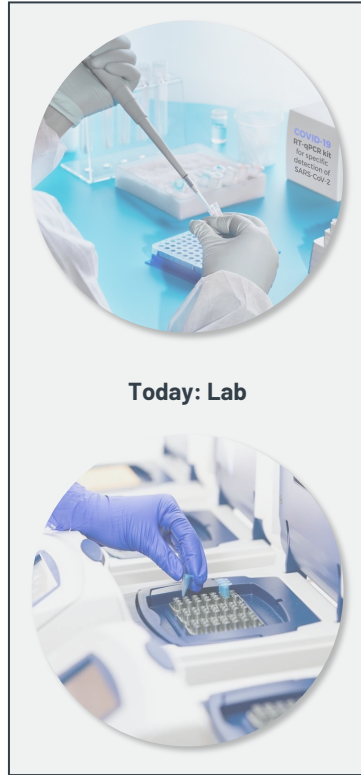
This additional effort takes 3 more hours out of staff and patient time

Clinics will significantly benefit by using our solution:


1. Results in less than 30 minutes
2. Optimal treatment planning
3. Testing at home to optimize flow in the dialysis clinics
4. Reimbursement for PCR test and for taking the test
5. Minimum effort
6. Mobile access for nephrologist

midge medical establishes test results in 15 min with reusable test device


Today: Lab

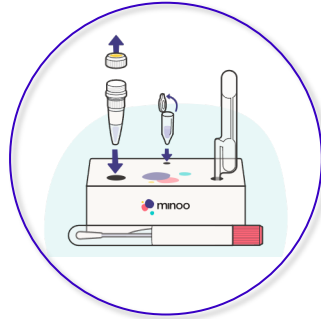Test kit
One simple pipetting step

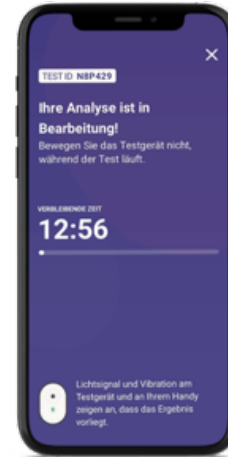Reader device Integrated multiple use
low-cost fluorometer (2 years lifetime)


Mobile app
User interface


Cloud back-end

- GDPR compatible Data analysis and aggregation
- interface to healthcare systems
- post-market surveillance

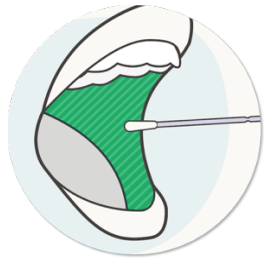Simple procedure for self-testing – fast and precise results

Take sample
Throat or nasal
swab

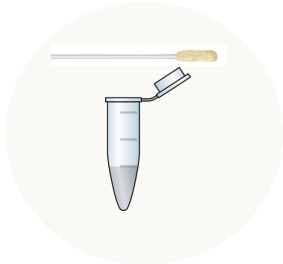
Process swab
Insert into lysis buffer,
mix with dry chemistry,
and transfer into test
container

Handling is similar to
antigen tests

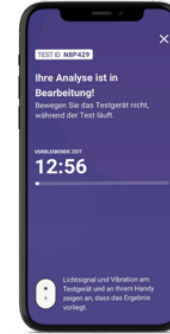**Place test container
onto device**
Sample is analyzed
and results are sent to
phone via Bluetooth

Get lab quality result
within 15 minutes

Digital monitoring of
process with
animated on-screen
instructions

Data connection
to cloud backend

Data aggregation for
healthcare systems
and possible real-time
disease mapping

Summary

midge medical provides a multiple-use handheld test device for molecular SARS-CoV-2 testing with PCR equivalent analytical sensitivity and selectivity

- **Test results in 15 mins**, with simple sample preparation by nurse or by patient under observation
- Test **simplifies the patient flow and creates a safe environment**
- **Reimbursement upside** for providing test to patients **assumed**
- **More assays** are available (respiratory: influenza, RSV, hemorrhagic fevers, ...)

Can you imagine using this test system in your clinic?

**We are looking for clinicians
to validate our test system in the clinic workflow,
to improve patient outcome,
to conquer the SARS-CoV-2 pandemic.**

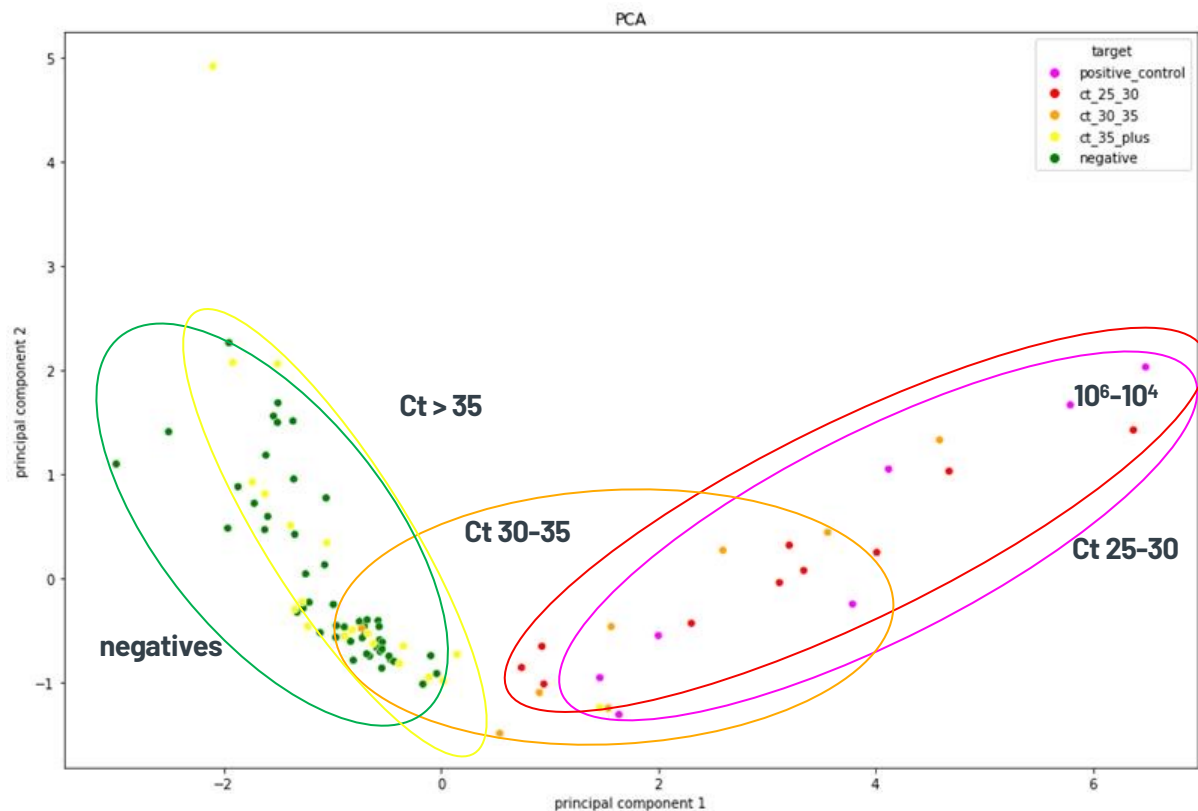
We love to hear from you.

midge medical GmbH
Colditzstraße 34-36, 12099 Berlin
info@midgemedical.com

Dr. Markus Riester
M +49 151 5350 9543
Markus.Riester@midge-medical.com

Clinical samples; LOD 8×10^2 copies/ reaction → equivalent to PCR up to CT=35

		Purification 1:2	
		PCR	RPA
Template volume		5µl/20µl	150µl/600µl
Total Dilution		1:8	
Device		RotorG	Minoo
CT 25-30	5555327357	25.74	+
	1941548797	26.27	+
	5562179767	26.49	+
	5562179197	26.74	+
	5015208497	26.92	+
	5015214997	28.1	+
	2079615717	28.53	+
	4263118057	28.99	+
	4263118007	29.08	+
	1909236637	29.15	+
CT 30-35	5555335137	30.57	+
	1202203387	32.05	+
	4269251867	32.96	+
	5562179037	32.98	+
	7502263737	33.22	-
	4269303787	33.32	-
	5562179837	34.3	+
	1939733757	34.53	+
CT 35-40	4269309297	35.5	+
	4269295417	36.35	+
	5015214747	36.69	-
	2079828497	37.27	-
	4263120457	38.07	-
	4269326397	38.46	-
	4269351807	38.49	-
	4269244557	38.51	+
	4269329867	38.79	-
	4269346527	38.98	+
	4269333017	39.5	-
	4269303777	40.02	-

Performance comparison Antigen vs. PCR vs. minoo test